Art Cast & Splint

Licensed by Korea-Prime

Iso 9001-2000

Iso 13485

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History :

Aiming to produce high-quality medical devices, in 2007 Zarrin Teb Kala established 2 factories manufacturing orthopedic fiberglass and polyester bandages licensed by Prime Company in Korea . After analyzing and carrying out researches , producing splint began as well . Presently in addition to supplying the domestic market , the company has been able to attract some customers in Asian and African countries leading to build a top brand in the world .The company is also the supplier of raw material in several factories inside and outside Iran.

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Sale Management :

Aala International Trade Company is the sole agent for selling Zarrin Teb Kala products inside and outside Iran. Using professional sales workforce, Aala has succeeded in making Art Cast, Top Cast, Aala Cast as popular and high-quality brands in domestic and foreign market.

Art Cast

Iso 9001-2000

Iso 13485

Art-Cast is comprised of fiberglass bandage impregnated with polyurethane resin.

Application instructions

- 1- Slip a vibril over the affected body part (for greater protection, use orthopedic padding)
- 2- Gloves must be worn
- 3- Open only one pouch at a time with gloved hands
- 4- Immerse the Art-Cast roll for 5~10 seconds in tepid water
- 5- Squeeze it 2-3 times so the setting time will be 4-5 minutes (warmer water makes the set time shorter)
- 6- Wrap spirally, taking care to avoid excessive tightness

Advantages:

- Strong and rigid
- Easy removal
- Excellent x-ray radiolucency
- Color range
- Light weight
- Cost-efficient
- Molding capability
- Air permeable
- Moisture resistant
- Superb adhesion

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
Iso 9001-2000

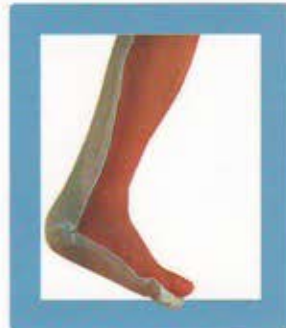Iso 13485

Art-Splint consists of a central layer which is covered by vibril on both sides. One side which covers the affected body part is thicker. This side features hydrophobic effect which starts to dry as soon as it comes into contact with water.

Application

- 1- Select the suitable size
- 2- Immerse the splint in lukewarm water(23-25[^]c)(the warmer the water, the shorter the setting time
- 3- Squeeze the splint tightly to remove the excess water
- 4- Slip the thicker part of the splint on the affected body part
- 5- Tie the part tightly with elastic bandage

Advantages :

- 1- Strength: rigid, strong, unbreakable
- 2- Light weight and comfortable
- 3- Flexibility and speed in application (No need of gloves)
- 4- Excellent Molding capability(in less than 5 minutes)
- 5- Superb x-ray radiolucency
- 6- Cost-efficient
- 7- Moisture resistant
- 8- Size ranges(flexibility in size)

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Maintenance conditions:

- Keep in a cool and dry place (15-30^c).
- Too high and low temperature reduces the longevity.
- Make sure the pouch is well-protected as minute holes leading to the exposure to any moisture or air cause rigidity
- Change the position of boxes every 3 months to prevent the resin inside from sediment.

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Fax No : +98-34-34287671

Sales office :

international AAla Trade Co. , Shahid Beheshti Street, Rafsanjan, Kerman,

Tel No:+983434265950

Fax No:+983434265950

www.aalaemc.com